Harmony Products Inc

HPI - 8199

Homecare Bed

User Manual

Contents

1.	Introduction	4
2.	Intended Use	5
3.	Product Description	6
	3.1 Model variations	6
4.	Symbols and Statements	7
5.	Safe Use Guidelines	8
	5.1 Safety information – nursing personnel, carers and users	8
	5.2 Safety information – bed assembly	9
	5.3 Safety information – maintenance and inspection	10
6.	Environmental Conditions	10
7.	Installation and Set-Up	11
	7.1 Assembling the bed	11
	7.2 Connecting the controller	14
	7.3 Before using the bed	14
8.	Handset Control	14
	8.1 10-button handset	15
9.	Bed Adjustments	15
	9.1 Knee-break section adjustment	15
	9.2 Manual calf section adjustment	16
	9.3 Backrest section adjustment	16
	9.4 Bed height adjustment	16
	9.5 Auto-contour adjustment	17
	9.6 Trendelenburg and Reverse Trendelenburg (on fitted models)	17
10.	Moving the Bed	17
11.	Selecting a Mattress	18
12.	Extending the Bed	18
	12.1 To extend the bed frame	18
	12.2 To extend the platform	18
13.	Accessories	19
	13.1 Attaching Side Rails to the bed	19
14.	Transport and Storage	20
	14.1 Disconnecting the controller	20
	14.2 Dismantling the bed	20
	14.3 Storing the bed	20

15.	Cleaning and Disinfection	20
15.1	Cleaning	21
15.2	Disinfection	21
16.	Maintenance and Inspection	21
16.1	Maintenance – daily	21
17.	Trouble Shooting.....	22
17.1	Potential faults	22
18.	Reuse.....	22
19.	Disposal	22
20.	Expected Service Life	23
21.	Specifications.....	23
21.1	Bed data.....	23
21.2	Electrical data	24
22.	Warranty	25
22.1	Product warranty	25
22.2	Conditions of warranty	25
22.3	The following are excluded from the warranty:.....	25
22.4	How to make a claim.....	26
22.5	General	26

1. Introduction

This User Manual contains instructions for the installation, use and maintenance of the *Harmony Products HPI-8199 Home Care Bed*.

The bed described in this User Manual is constructed to the following standards:

- *EN 60601-2-52*

This operating manual will familiarize you with the functions of the bed and in addition describes:

- How to set up the bed
- How to operate the adjustment controls
- How to look after and maintain the bed

Caution: Harmony Products shall not be liable for any damage or injury caused by failure to follow the proper instructions as described in this User Manual.

Caution: Before using the bed all staff must familiarize themselves thoroughly with the various parts and controls as detailed in this User Manual.

This User Manual has been drawn up using information available at the time of printing with regard to the design and operation of the bed.

We reserve the right to make changes due to technical improvements.

If you have any questions regarding the bed, please contact *Harmony Products* (contact details are provided on back cover).

Note: *Harmony Products* reserves the right to modify the information in this UserManual at anytime. The information in this User Manual may vary slightly with respect to the basic design of the product.

2. Intended Use

The bed is intended to support patients using the moveable sections as and if required within the following environments;

- Nursing home
- Home

This bed must not be used for any other purpose than that which it is intended, for example it must not be used as a standing platform. It is for indoor use only.

This bed is not suitable for children under 12 years of age.

Before using the bed a risk assessment should be performed. This should include but should not be limited to:

- The risk of trapping or pinching
- The risk of the patient slipping between mattress and the side rails
- Unintended operation of the bed
- Any limitation of the patient's ability to operate the bed

Thoroughly read the entire User Manual prior to setting up and using the bed.


Ensure that:

- The entire User Manual is read by all persons entrusted with the setting up, operation or maintenance of the bed.
- All persons entrusted with the setting up, operation or maintenance of the bed have access to this User Manual at all times.

Any damage resulting from non-observance of this User Manual is excluded from the warranty.

The illustrations contained in this User Manual show examples of how to handle the *Harmony Products Home Care Bed* and may differ from your particular model.

3. Product Description

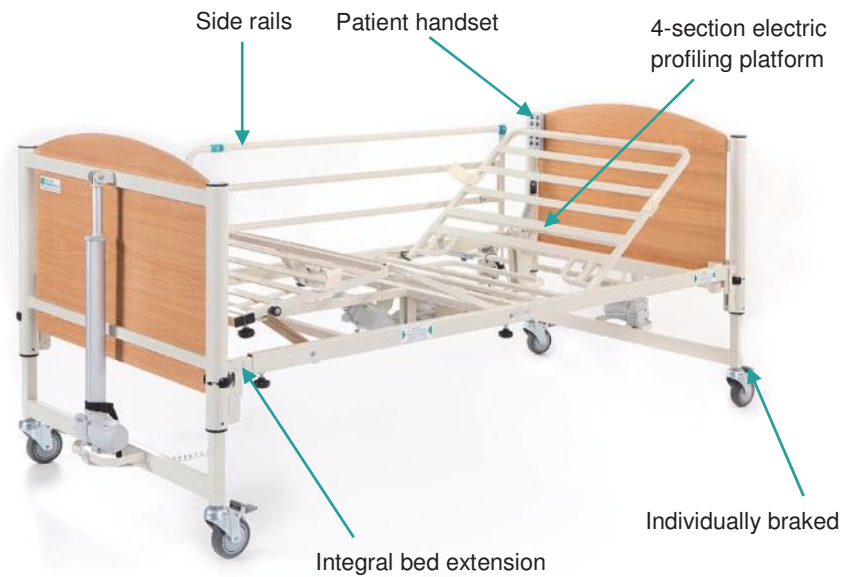Note: The above illustration is for reference purposes only. The specification of your actual **Home Care Bed** may vary to the above illustration.

- 4-section electric profiling with variable height control and adjustment
- Active auto-regression profiling reduces shear and friction in sacral area (*selected models only*)
- Integral bed extension of 170mm
- Reverse Trendelenburg - variable tilt positioning (*selected models only*)
- Handset features a swipe fob lockout
- Transport brackets attach to the bed to help prevent loss
- Maximum user weight 180kg – Safe working load 200kg
- Accessories – please see *Accessories Section 13* in this Manual.

3.1 Model variations

HPI-8199 (10-button handset)
Bed raise/lower function
Head / Foot Incline
Lock-out function
Reverse Trendelenburg
Trendelenburg
(Optional)
Battery back-up (retro-fit possible)

Note: Fitting of accessories must be made in conjunction with the User Instructions supplied with the accessory.

Note: The fitting of accessories may present additional risks to Users and suitable precautions should be taken.

4. Symbols and Statements

This User Manual contains the following symbols which are used to show you specific risks when dealing with the product or information on how to simplify handling.

Caution: This symbol indicates the need for the user to consult the instructions for use for important cautionary information such as warnings and precautions that cannot, for a variety of reasons, be presented on the medical device itself.

Symbols used in this manual and on the *Home Care Bed*:

B symbol, indicates this product is according to degree of protection against electric shock for type B equipment.

Caution, read the instructions before use.

IPX4

Protection type

Disposal of electrical and electronic equipment (WEEE): This product should be handed over to an applicable collection point for the recycling of electrical and electronic equipment.

CE certification in accordance with 93/42/EEC.

Protection class II (Protective insulation).

Note: Maximum continuous operation of the actuator motors is 2 minutes at maximum load; it must then be followed by a period of rest for 18 minutes, before being activated again.

5. Safe Use Guidelines

The bed is fitted with a locking handset; which is achieved via a locking magnetic fob.

Caution: When using any equipment that extends under the bed ensure there is no risk of entrapment, pinching of cables or risk of damage to motors or actuators when the bed is moved to its lowest settings.

Caution: Although this bed has been designed to be used with the common types of portable hoists care should be exercised when using this bed with all portable hoists. The user must ensure that sufficient clearance is afforded to all hoist legs, masts and booms and that cables of the bed and hoist are not trapped, snagged, kinked or pinched when operating the bed or the hoist.

5.1 Safety information – nursing personnel, carers and users

Caution: Risk of Accidents!

- Do not use electrical medical appliances on patients in these beds involving intravascular or intra-cardiac connection.
- Do not use the bed in the vicinity of strong electromagnetic fields.
- Always plug the bed mains plug into a separate wall socket. Do not use a multi-plug adaptor.
- Route the mains cable to the wall socket in such a way that there is no mechanical tension, and the mains cable cannot be pinched, kinked or run over.
- When routing cables from other equipment within the bed, precautions should be taken to avoid squeezing those parts of the bed.
- Please observe the respective equipment operating and assembly instructions with regard to any additional safety risks that may arise when using the bed in conjunction with other equipment.

Warning: Danger to life and risk of injury due to trapping/pinching!

- When carrying out adjustments, watch the occupant in the bed and the bed's surroundings.
- Ensure that underweight/very thin patients cannot slip down between the mattress and side rails.
- Ensure that the side rails are properly locked in place.
- When lowering the bed make sure that there are no obstructions under the bed; pay particular attention to the feet of the person operating the bed and those of any other person in attendance.

Warning: Possible danger to life due to alteration of the patient's position

- Always lock (disable) all handset adjustment functions on the handset which could cause danger to the patient's health if adjusted (consult doctor).
- Always take account of the patient's condition when locking (disabling) the adjustment options.

Warning: Accident hazard due to falling out of the bed!

- The bed should always be left in its lowest position when the patient is unsupervised or for the patient to get in or out of the bed in order to reduce the risk of injury due to falls.

Caution: Risk of short circuit and fire

- Before moving the bed, disconnect the mains plug and secure the mains cable from being run over.
- Do not use the bed in surroundings in which flammable gases or vapors (e.g. from anesthetics) may be present.

Caution: Danger due to actuator motor overheating!

- Before adjusting the bed height, ensure that nothing is blocking the full adjustment movement.
- Avoid operating the actuator motors continuously for more than two minutes.
- Maximum continuous operation of the actuator motors is 2 minutes; the minimum pause after 2 minutes operation is 18 minutes.
- Never use more than two actuator motors simultaneously.

5.2 Safety information – bed assembly

Warning: Risk of Injury due to incorrect assembly!

- Ensure that all bed components are correctly assembled.
- Check that all adjustment options are in perfect working order after assembly,

Caution: Risk of Accidents!

- Only perform assembly work using suitable tools.
- Only perform the work described if you are familiar with handling the required tools.

5.3 Safety information – maintenance and inspection

Only allow maintenance and inspection work to be carried out by or under the supervision and guidance of trained expert personnel who, on the basis of their specialist training, knowledge and experience - in addition to knowledge of all the relevant regulations - are capable of evaluating the condition of the bed and of recognising any possible effects and hazards.

Always read the relevant chapters of this User Manual thoroughly before carrying out maintenance and inspection work.

Any damage resulting from non-observance of this User Manual is excluded from any liability of the manufacturer.

Caution

- Always carry out a function test following maintenance work and inspections.
- When operating the actuator motors, ensure that the bed cabling cannot be pinched, overstretched or kinked.

Caution: Hazards due to electrical and mechanical faults!

- Immediately remove faulty/defective beds from operation and secure them against unauthorized use. Disconnect the mains plug from the wall socket.
- Only use original spare parts from *Harmony Products*.
- Always carry out the prescribed maintenance and inspection work at the specified intervals, see *Section 16 Maintenance and Inspection*.
- Do not carry out any alterations to the electrical or mechanical fittings supplied with the bed unless authorized to do so by the manufacturer. Unauthorized alterations lead to any liability of the manufacturer becoming null and void.
- Only perform maintenance and inspection work using suitable tools.
- Do not open electrical components.

6. Environmental Conditions

	Min	Max
Operating temperature range	10 ⁰ C	40 ⁰ C
Relative Humidity (RH)	10%	60%
Altitude range (above mean sea level)	0m	3,000m

The IP (liquid and dust ingress) rating for the control module on the *Home Care Bed* is IPX6.

The *Home Care Bed* are designed for medical purposes only. Electrical installations must comply with your local electrical installation requirements.

7. Installation and Set-Up

7.1 Assembling the bed

Remove packaging; all cardboard is recyclable; carefully dispose of all plastic packaging to avoid risk of suffocation; recycle plastics where facilities exist.

Caution: Read these instructions carefully before attempting to assemble the bed.

1. Before starting to assemble the bed ensure that the brakes are engaged on all wheels. These must be foot operated only.
2. Ensure all elevating sections of the bed are secured to the frame, to prevent entrapment injury.
3. Remove the R-clips, the plastic spacer and then the pins from the foot section of the bed (*Figure 1*); retain pins, spacer and clips for later use.
4. Carefully remove the head section from the transport bracket and place it motor side down on the floor.
5. Loosen the thumb wheels, if removed ensure they are retained for later use.
6. Carefully remove the foot section from the transport bracket and place motor side down on the floor.
7. Rotate the locking collars (*Figure 2*) retaining the transport bracket to reveal the sticker on one bed end and carefully remove the bed end from the bracket; allowing the remaining bed end to lean against a wall.

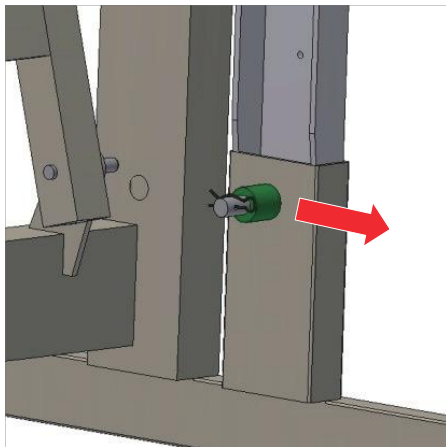
Figure 1: Removal of R-clips

Figure 2: Collar unlocked

8. Lift the foot platform section onto the bed end (*Figure 3*) and clip into the bracket, ensuring that the bed end is fully clipped in place.
9. Rotate the locking collar to cover the label (*Figure 4*). Repeat both sides.

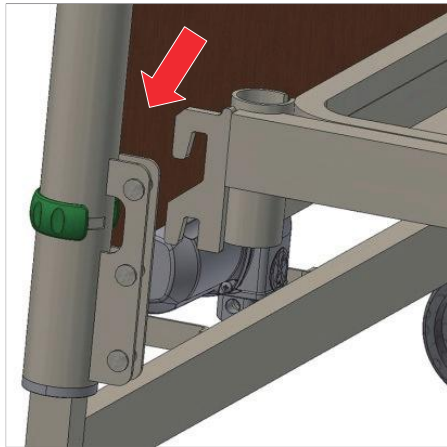
Figure 3: Foot section attachment to bed end

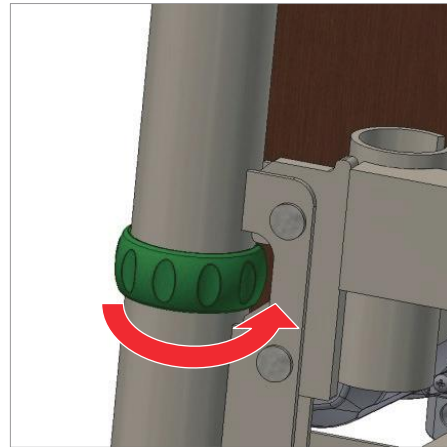
Figure 4: Collar locked

10. Remove the transport bracket from the second bed end by rotating the locking collar to reveal the label (*Figure 2*).
11. Lift the bed end to the end of the head platform section.
12. Lift the head platform section onto the bed end and clip into the brackets (as in step 8); ensuring that the bed end is fully clipped in place.
13. Rotate the locking collar to cover the label (*Figure 4*). Repeat both sides.
14. Release the castor brakes on one end of the bed only.
15. Carefully align the two sections of the bed and slide together (*Figure 5*)
16. Tighten the mid-section thumb wheel and insert the locking pin, plastic spacer and R-clip (*Figure 6*). Repeat both sides. Re-apply the castor brakes.

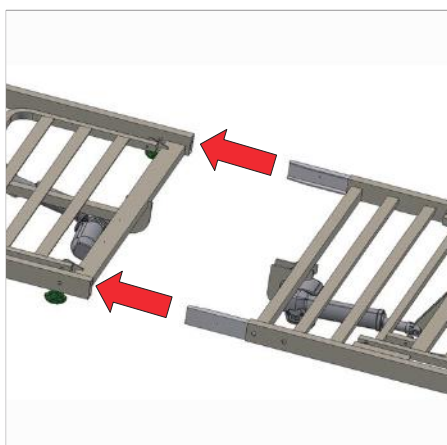
Figure 5: Platform connection

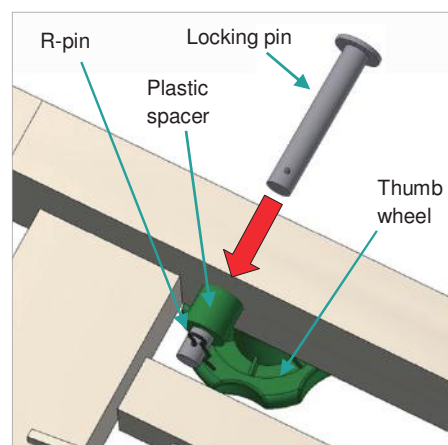
Figure 6: Securing platform sections

17. The transport brackets can be safely stored on the bed frame to prevent loss of the brackets. To do so, raise the bed to its full height. The transport brackets can then be attached to the fixing points on the underside of the bed as indicated (Figure 7). Ensure that each bracket is locked in place.
18. Attach the 4 mattress retaining clips to the bed frame (Figure 8) at the locations shown (Figure 9).
19. Remove the retaining straps holding the moving parts of the frame to the bed ends at the locations shown (Figure 9).

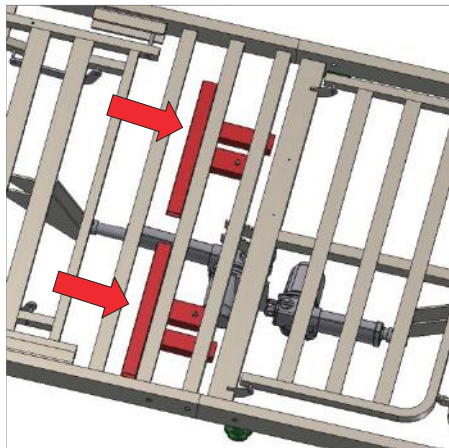
Figure 7: Transport bracket storage

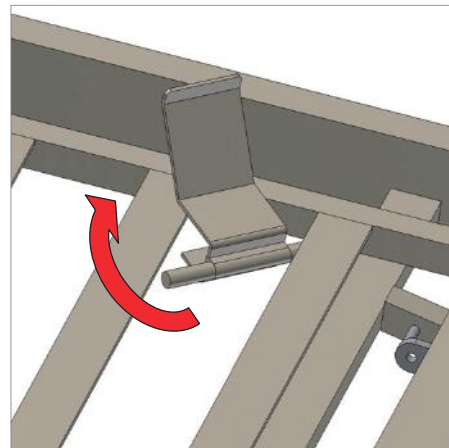
Figure 8: mattress retainer

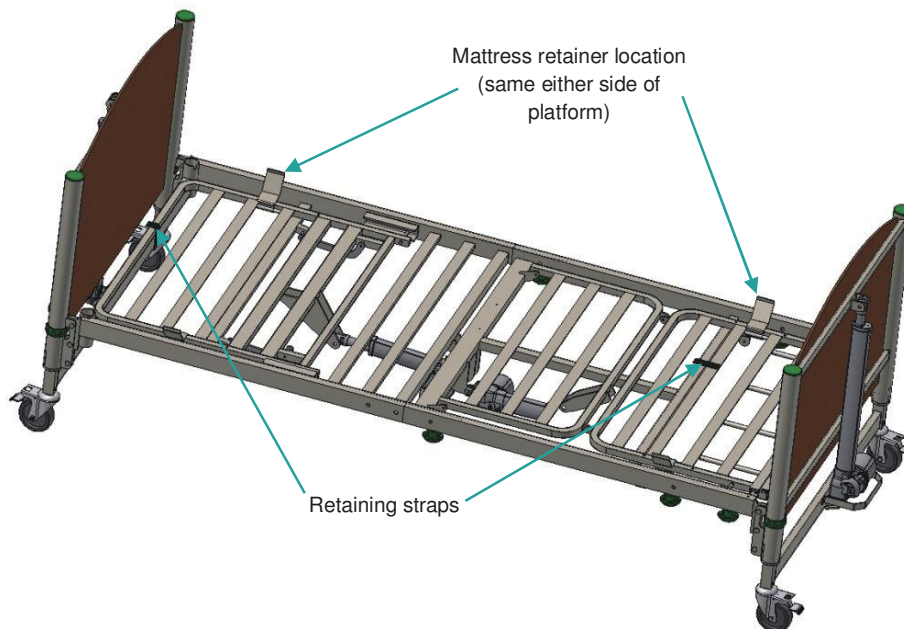
Figure 9: Location of mattress retaining clips and retaining straps

7.2 Connecting the controller

Once the bed is assembled the actuators and the controller can be connected.

1. Ensure all cabling is carefully routed under the bed; avoiding any risk pinching the cables.
2. The connectors to plug into the controller are all colour coded, as are the sockets on the controller (*Figure 10*).
3. Connect the sockets on the actuators.
4. Connect the Handset.

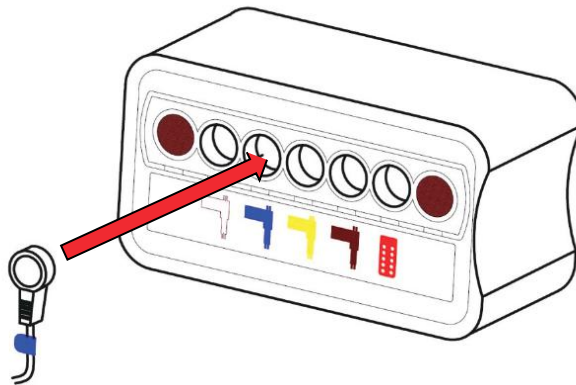
Figure 10: Connecting the controller

7.3 Before using the bed

1. Check all locking collars are locked, i.e. the label is covered.
2. Ensure all retaining straps are released.
3. Check all pins, plastic spacers and R-clips are secure.
4. Check all thumb wheels are tight.
5. Are all cables correctly routed and have sufficient slack to allow for movement of the bed?
6. Check there are no obstructions that may affect the bed movement.
7. Apply the castor brakes.
8. Has a risk assessment been undertaken?
9. Connect the bed to the mains supply and test.

Caution: Ensure all cables are not under any undue tension when at their most extended and are free from any moving parts of the bed.

8. Handset Control

Handset Controls are supplied with buttons.

Caution: Ensure that no person, limbs or objects in any way interfere with the desired movement of the bed.

8.1 10-button handset

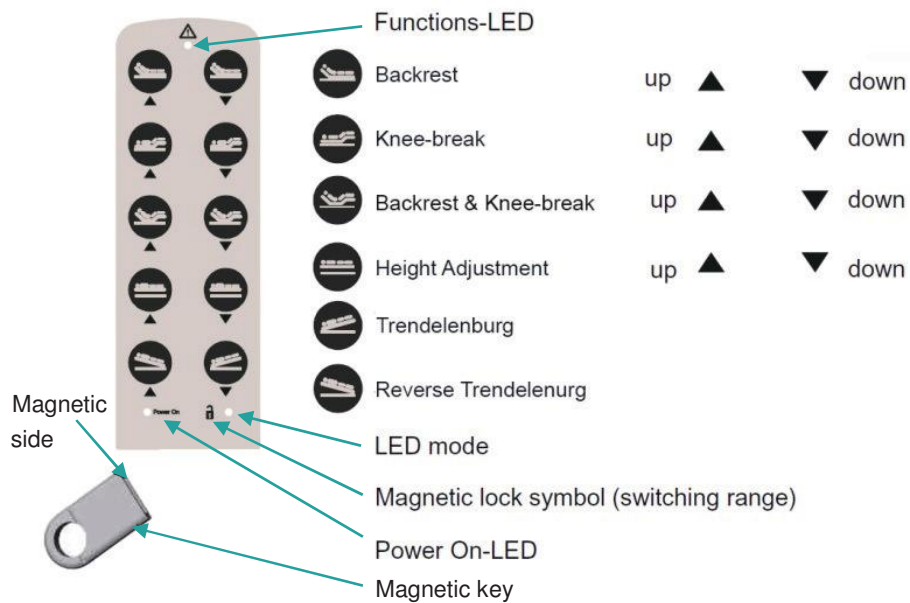
Figure 12: 10-button handset

Functions of the locking device

By passing the magnetic side of the magnetic key over the padlock symbol, three modes can be released or blocked.

- To put the handset into operation, connect it to the control unit. The keypad of the handset is completely blocked. The LED next to the padlock symbol does not glow.
- Passing the magnetic side of the magnetic fob once over the padlock symbol releases the handset adjustment functions except the Trendelenburg function. The LED next to the padlock symbol glows green.
- Passing the magnetic side of the magnetic fob over the padlock symbol once again, enables the Trendelenburg function. The LED next to the padlock symbol glows orange.
- If no key is pressed after approximately 45 seconds of the handset not being used it will automatically lock.

Caution: Keep the handset away from magnetic objects and strong magnetic fields as this may cause the handset to lock or unlock.

9. Bed Adjustments

9.1 Knee-break section adjustment

To raise the knee-break section of the bed, press and hold the Up Knee-Break Adjustment Button on the handset to reach the desired position.

To lower the knee-break section of the bed, press and hold the Down Knee-Break Adjustment Button on the handset to reach the desired position.

9.2 Manual calf section adjustment

The bed is fitted with a manually adjustable calf/leg section. When the knee-break function on the handset is operated the height or angle of the calf/leg section can be adjusted.

- To decrease the knee break angle, manually raise the calf/leg section to the desired position. The calf/leg section will automatically lock into place (*Figure 13*).
- To lower the calf/leg section (from raised height), fully raise calf/leg section manually and then gently lower it down (*Figure 14*). Then press and hold the Down Knee-Break Adjustment Button on the handset.

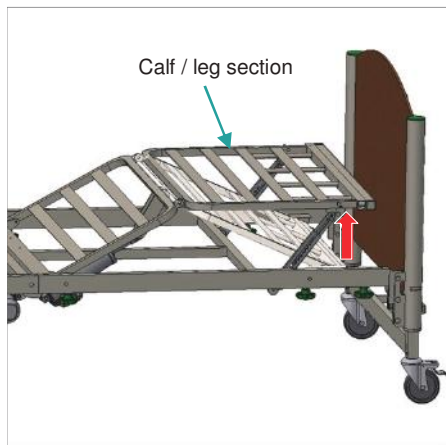
Figure 13: Raising manual calf section

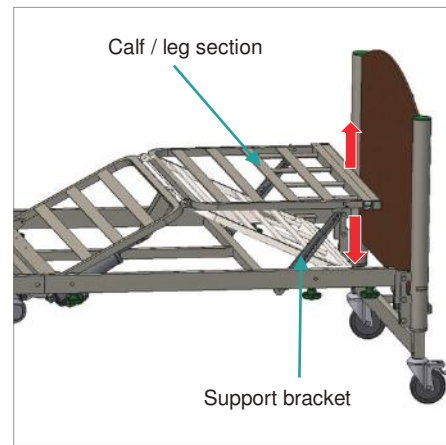
Figure 14: Lowering manual calf section

Caution: Before lowering the knee-break section, ensure that the calf section is set to its lowest position.

9.3 Backrest section adjustment

To raise the head section press and hold the Up Backrest Adjustment Button on the handset until it reaches the desired height.

To lower press and hold the Down Backrest Adjustment Button on the handset.

9.4 Bed height adjustment

To raise the height of the bed press and hold the Up Height Adjustment Button on the handset until it reaches the desired height.

To lower press and hold the Down Height Adjustment Button on the handset.

Caution: Before adjusting the height of the bed ensure there is nothing to obstruct the movement of the bed and ensure that no cables will become pinched or be over stretched.

9.5 Auto-contour adjustment

Auto-contour adjustment allows for the simultaneous adjustment of both the backrest and knee-break sections.

To raise the backrest and knee-break sections of the bed, press and hold the Up Auto-Contour Adjustment Button on the handset to reach the desired position.

To lower the backrest and knee-break sections of the bed, press and hold the Down Auto-Contour Adjustment Button on the handset to reach the desired position.

9.6 Trendelenburg and Reverse Trendelenburg (on fitted models)

To tilt the mattress platform into the head down tilt (Trendelenburg) press and hold the Trendelenburg Adjustment Button.

To tilt the mattress platform into the foot down tilt (Reverse Trendelenburg) press and hold the Reverse Trendelenburg Adjustment Button.

Caution: Before carrying out these operations, check there are no objects on or under the lower frame which could interfere with the bed movement. These functions can be locked out by means of the magnetic key fob on the handset.

10. Moving the Bed

Before moving the bed, ensure that a risk assessment is completed.

To move the bed:

- Release all 4 castor brakes, these are foot only operated.
- Move the bed to the desired location.
- Taking care not to run any cables over.
- Re-apply the castor brakes.
- Check there are no obstructions near the bed.
- Check the cabling is not pinched or kinked anywhere.
- Re-connect the bed to the mains supply.

Caution: Ensure there is no risk of injury to any person involved in moving the bed and ensure there is no risk of damage to the bed.

11. Selecting a Mattress

Ensure the selection of an appropriate mattress depending on whether the bed is set to standard or extended mode. Having placed the mattress on the bed, a risk assessment should be performed to identify and potential entrapment risks for the user. Note that, if using side rails, the distance from the top of the uncompressed mattress to the top of the side rail should be more than 220mm.

12. Extending the Bed

The bed can be extended by 170mm. Adjustment is made at the foot end of the bed.

12.1 To extend the bed frame

- Ensure bed frame moving parts are all flat. (see Section 9 for bed adjustments)
- Lower the bed to its minimum height.
- Apply brakes to head end of the bed.
- Loosen the 4 thumb wheels at the foot end of the bed (2 per side)
- Loosen the 2 thumb wheels on the bed extension piece.
- Unlock the foot end castors.
- While holding the foot end near the locking collars pull the bed extension out until it reaches the end stops (Figure 15).
- Re-tighten all the thumb wheels and secure the actuator cable back into its clip.

12.2 To extend the platform

- Loosen the 2 hand wheels located on the platform section.
- Slide the platform extension out until it reaches the end stops (Figure 16).
- Re-tighten the extension thumb wheels (be careful not to over tighten as damage to the extension tubes may occur).

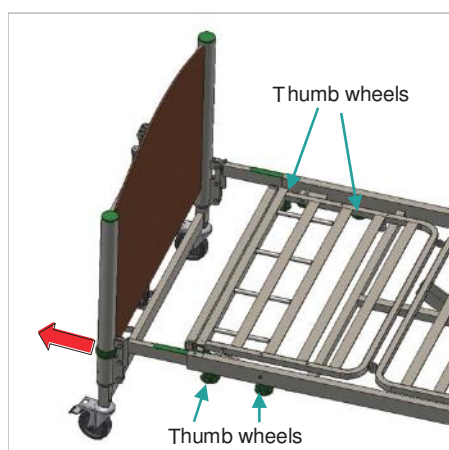
Figure 15: Extending the bed frame

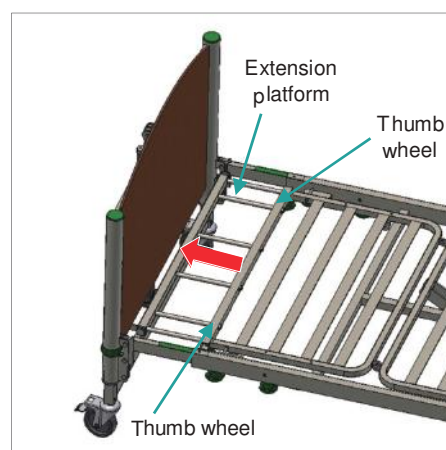
Figure 16: Extending the platform

Caution: Ensure that the mattress platform has been fully extended to bed end before use.

13. Accessories

There are some accessories available for the bed provided by *Apex Health Care Products* and have been tested with this bed. Below is a list of the accessories available for the bed.

Product Code	Description
APC-8199 HSR	Home Care Bed Half Side Rails (option)
APC-8199 FSR	Home Care Bed Full Side Rails (option)
APC-8199 BATTERY	Home Care Bed Backup battery kit (can only be retro-fitted by manufacturer or an authorised Service Agent, option)

All side rails listed above meet the requirements of the MHRA guidelines DB 2006(06) v2.0. It is strongly recommended that these guidelines are read prior to using side rails.


Caution: *If any other accessories are attached to the bed a full risk assessment must be completed.*

13.1 Attaching Side Rails to the bed

When attaching side rails the consideration of safety is paramount. Side rails with the product codes APC-8199 HSR and APC-8199 FSR have been specially designed for the *Home Care Bed*; however you must ensure the following points are adhered to:

- Position the Side Rails according to the labels on the bed and side rails.
- Side Rails can be installed at the Foot Section either, with the distance not less than 580mm from the bed end.
- The gap between the side rail and the bed must less than 60mm or more than 318mm to minimise the risk of entrapment and to comply with entrapment guidance and regulations.
- Ensure that the distance from the top of the uncompressed mattress and the top of the side rail is at least 220mm.
- Screw in the screw knob to securely fix the side rail to the bed.
- Ensure that the rails are securely fitted before use every day.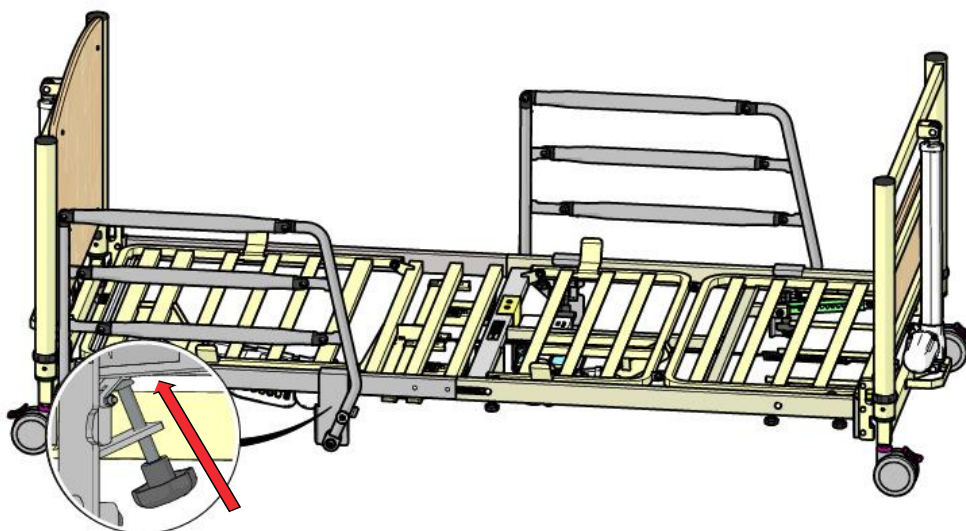

14. Transport and Storage

14.1 Disconnecting the controller

1. Ensure castor brakes are engaged – these are foot operated only.
2. Ensure the mains lead is disconnected before starting to dismantle the bed for transport or storage.
3. Remove the mattress.
4. Remove the connectors from the controller.
5. Remove cables from cabling system where they pass from one side of the bed to the other.

14.2 Dismantling the bed

1. Remove any accessories attached to the bed.
2. If the bed has been extended; return it to the standard length.
3. Apply the castor brakes; this is a foot only operation.
4. Remove the mattress retaining clips.
5. Remove the transport brackets from the bed by pressing the release button.
6. Attached the retaining straps to the elevating parts of the bed; to avoid entrapment risk.
7. Remove the R-clips, plastic inserts and pins from both sides of the mid-section of the bed.
8. Release the thumb wheels; if removed stored carefully for re-use.
9. Release the castor brakes on one end of the bed only.
10. Carefully slide the two halves of the bed apart and rest them on the floor.
11. Apply the castor brakes; these are foot only operated brakes.
12. Unlock the retaining collars on one end of the bed; if the label is visible the collar is unlocked.
13. Carefully unclip the end of the bed; taking care not to allow it to fall to the floor.
14. Repeat for the other end of the bed.
15. Clip the head and foot end together using the transport bracket.
16. Turn the locking collars to hide the labels – this locks the transport bracket in place.
17. Place the head end of the bed onto the transport bracket and secure using the thumb wheels.
18. Place the foot end of the bed onto the transport bracket and secure by replacing the pins, plastic retainers and the R-clips.
19. Replace the cardboard packaging if retained.

14.3 Storing the bed

When not in use this bed must be stored in a dry, clean environment. We recommend that storage temperatures are between -5°C -40°C and that % Relative humidity (RH) is between 10~60%.

15. Cleaning and Disinfection

Disconnect the bed from the electricity supply before carrying out decontamination procedures.

15.1 Cleaning

- Please ensure that the power plug is removed from the mains socket prior to attempting any cleaning procedure. This bed is not provided with a mains switch so the mains plug is the only separation from the mains.
- Clean using a damp cloth.
- Only use mild detergents without scouring agents to clean all metal parts and wooden and plastic surfaces.
- Never use sharp edged tools (knives, metal scrapers) or aggressive solvents for cleaning.
- Never use high-pressure cleaners for cleaning.
- Observe the instructions on how to use the detergents in order to avoid damage to the surfaces.
- The bed should be carefully dried after cleaning and before plugging it back in to the mains.

15.2 Disinfection

- Disinfect using a soft cloth.
- Only use normal household disinfectant.
- Do not use harsh, abrasive disinfectant / cleaners.
- Observe the instructions on how to use the disinfectant in order to avoid damage to the surfaces and to protect your health.

16. Maintenance and Inspection

Hazards due to electrical and mechanical faults!

- Immediately remove faulty/defective medical beds from operation and secure them against unauthorised use (disconnect the mains plug).
- Immediately show any defects to the responsible personnel.
- It is imperative that you always rectify any faults which could influence the function and safety of the nursing bed immediately.
- Observe the safety instructions contained in Section 5.

16.1 Maintenance – daily

- The daily maintenance work can be performed by the nursing personnel or carer:
- Lower both bed end actuator motors until they switch off automatically.
- Check the side rails (option) for easy movement and damage.
- Check the correct function of the locking device on the side rails (option).
- All moving parts on this bed are fitted with maintenance-free self-lubricating bearings and should not be oiled or greased.
- Check to ensure that all cables are located safely, not dangling down, and not at risk of being caught in the bed mechanism, by a mobile hoist or any other equipment that may be used under the bed.

Whenever the bed is moved

- Check the mains cable and plug for damage and kinks.
- Route the mains cable in such a way that it cannot be pinched or rolled over.
- Check to ensure that all cables are located safely, not dangling down, and not at risk of being caught in the bed mechanism, by a mobile hoist or any other equipment that may be used under the bed

17. Trouble Shooting

17.1 Potential faults

The table below has been created to assist with resolving faults you may come across; if the instructions here do not resolve the issue or the issue you are experiencing is not listed please contact Ultimate Healthcare using the details found on the back cover of this manual.

Problem	Possible Cause	Resolution
<ul style="list-style-type: none">LED glows when the handset is lockedLED does not glow when the handset is unlocked	<ul style="list-style-type: none">Fault in the handsetFault in the controller	<ul style="list-style-type: none">Contact Ultimate Healthcare using the details on the back cover of this manual
<ul style="list-style-type: none">No adjustment of the bed can be made when the power is ON	<ul style="list-style-type: none">Handset is locked (after a power failure the handset automatically switches to the locked state)	<ul style="list-style-type: none">Release the handset using the magnet
<ul style="list-style-type: none">The function-LED does not glow when a key is pressed	<ul style="list-style-type: none">Check the handset is unlockedControl unit or handset defective	<ul style="list-style-type: none">Contact Ultimate Healthcare using the details on the back cover of this manual
<ul style="list-style-type: none">The handset doesn't function – no power light	<ul style="list-style-type: none">Mains power not connectedHandset not connected to the controller	<ul style="list-style-type: none">Connect to the mains powerConnect the handset to the controller; following the colour coding
<ul style="list-style-type: none">When a button is pressed on the handset, the corresponding actuator does not respond	<ul style="list-style-type: none">Connectors not plugged into the correct socketsFaulty actuator	<ul style="list-style-type: none">Check the connections, these are colour codedContact Ultimate Healthcare using the details on the back cover of this manual
<ul style="list-style-type: none">An unexpected operation occurs when a button is pressed on the handset	<ul style="list-style-type: none">Connectors not plugged into the correct sockets	<ul style="list-style-type: none">Check the connections, these are colour codedContact Ultimate Healthcare using the details on the back cover of this manual

18. Reuse

The *Apex Health Care Products* beds are designed for reuse. They must be cleaned and disinfected prior

to reuse as per Section 15.

19. Disposal

Beds consist of:

- Metal parts
- Plastic parts
- Electronic components
- Electric cables
- Rechargeable batteries/batteries

Disposal of the individual materials must be carried out in accordance with the valid regional environmental and disposal regulations.

Beds are not to be disposed of with household waste; they must be taken to local waste collection depots.

Electronic components, electric cable and rechargeable batteries/batteries are not be disposed of with household waste but taken to the stipulated waste disposal depot and disposed of in accordance with the current regulations.

Note

If you have any questions regarding disposal please consult your local waste disposal depot or *Apex Health Care Products*.

20. Expected Service Life


The expected service life of any medical bed is highly dependent on usage, storage conditions and maintenance regime. Providing that usage is not excessive, that the bed is used and stored in ideal conditions and that the routine maintenance is strictly followed, that any damage is repaired immediately then this bed is expected to have a service life in excess of 5 years.

21. Specifications

21.1 Bed data

Overall length	2233mm
Overall length (extended)	2403mm
Overall width	946mm
Mattress platform	1982 x 883mm
Height range	234 - 638mm / 294 - 698mm
Backrest angle	0-72 ⁰
Knee-break angle	0-25 ⁰
Calf raise angle	0-18 ⁰
Trendelenburg	12 ⁰
Reverse Trendelenburg	12 ⁰
Safe Working Load (SWL)	200kg
Max patient weight	180kg
Total bed weight	86kg
Individual bed section weights:	
Head platform section	25kg
Foot platform section	25kg
Bed ends	18kg (each)

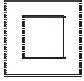21.2 Electrical data

	Do not dispose of in household waste
	Voltage type - alternating
	Insulation Class II, Type B
	The patient is not separated from the Ground and the chassis
	For indoor use only
100V - 240V	Voltage supply
50/60Hz	Voltage input frequency
3.15A (control pack) 3.50A (actuators)	Current drawn by device
Intermittent (periodic motor operation)	Maximum continuous operation of the actuator motors is 2 minutes at maximum load; It must then be followed by a period of rest for 18 minutes, before being activated again.
40dB	Sound Level
IPX6 – Control Pack IPX4 –Actuators	Protection Class

22. Warranty

Harmony Products is committed to selling high-quality products. However for one reason or another there may be a time you need to exchange or repair a product. Please make sure you check all items carefully on receipt and follow the manufacturer guidelines. It is recommended you keep a receipt or order number to ensure any warranty claim is handled efficiently. NOTHING IN THIS WARRANTY AFFECTS YOUR STATUTORY RIGHTS

22.1 Product warranty

- Products sold by *Harmony Products* (the "Company") are warranted against faulty materials or workmanship under normal use. The Warranty is for **Four Years** on the motors/electronics, and **Limited Lifetime** on the frame.
- For details of the warranty offered for your Product please call our customer services on **1-800-265-9615**.

22.2 Conditions of warranty

- Unless otherwise agreed in writing by the Company the Warranty applies to the original registered purchaser of the Product (the "Customer" "you" or "your") and this Warranty only covers defects that occur to the Product during the Warranty Period.
- Defects that occur to the Product during normal usage will, at the sole discretion of the Company, be replaced or repaired free of charge during the Warranty Period [or have parts replaced free of charge], including parts and labour. For the Warranty to be valid the Customer must ensure that:
 - a. the Customer notifies the Company within the Warranty Period of any defect and provides evidence of the date of purchase of the Product, your customer number and the Product Number;
 - b. unless otherwise agreed, the Customer permits the Company onto the Customer's premises to carry out any necessary inspection of the Product(s) and/or repairs;
 - c. all inspections and/or repairs are carried out by the Company or the Company's authorised agent;
 - d. the Product has been used for its intended purpose;
 - e. repairs to the Product have not been carried out or attempted by the Customer;
 - f. the Product has not been damaged by change, modification or use of the Product, which in the sole judgement of the Company affects the performance, stability or purpose for which it was manufactured.

22.3 The following are excluded from the warranty:

- Repairs due to breakdown caused by use other than domestic by you or your family, or by deliberate damage or neglect of the Products and (where provided) failure to follow the Product care instructions as provided by the Manufacturer.

- a. cosmetic damage and/or non-functional parts which do not affect the normal use of the Products including but not limited to scratches and rust;
 - b. damage caused by fair wear and tear;
 - c. work which relates to a manufacturer's recall of the Products;
 - d. any loss suffered as a result of not being able to use the Product, or any loss other than the repair or the replacement cost of the Product.
- Before agreeing to replace or repair your Product under this Warranty, the Company reserves the right to require that it is returned to the Company.
 - The Company regrets that when repairing or replacing your Product under this Warranty, the Company must reserve the right to use alternative specifications or materials, which may not match other items in a set.
 - The Company reserves the right to charge for the expense of visiting you when no fault has been found with the Product after the Company has inspected it.

22.4 How to make a claim

If you wish to make a claim under the Product Warranty please contact the Company on **1-800-265-9615**; e-mail us at: **info@harmonybeds.ca** or contact us by mail to **740 Huronia Road, Unit 7, Barrie, Ontario L4N 6C**, giving full details of the defect, your name, address and the date you purchased the Product. You are required to provide proof of the date you purchased the Product.

- a. The Company will discuss with you whether a repair or replacement will be offered. The option of a repair or replacement is at the Company's sole discretion.
- b. Where a repair to the Product is agreed, the Company will decide at their sole discretion whether the required repair work will be carried out at your premises or at the Company's premises.
- c. Where the Product is to be repaired at the Company's premises provided you live within Barrie *Ontario*, the Company will collect your Product from you and return it to you once it is repaired free of charge.
- d. Where your Product is to be replaced, provided you live within Barrie *Ontario*, the Company will deliver a new Product to you free of charge.
- e. If you live outside *Barrie Ontario*, you will be required to pay a delivery charge of the collection, and/or delivery of your Product.

22.5 General

- a. This Warranty applies to the original registered purchaser of the Product only and may not be transferred.
- b. This Warranty will be governed by *Canadian Law*.

Harmony Product Inc.
740 Huronia Road, Unit 7
Barrie, Ontario L4N 6C^
Tel: 1-800-2659615
Email: info@harmonybeds.ca
<https://harmonybeds.ca>

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